MOTIN CATHETERS

CE 0123

	Materials	References	Radio-opacity X.R.O Line	Characteristics
Motin suction catheters	PVC	06SM21xx	No	xx (depends on references): 10, 12, 14, 16 or 18 Lengths: 65 cm Terminal orifice with lateral eye Straight Protective sleeve No mark

DESCRIPTION

PVC suction catheter with polyethylene protective sleeve

INDICATION

To clean the secretion from the respiratory tract

CONTRAINDICATIONS

Not to be used in patients with known hypersensitivity to any of the material used

PRECAUTIONS

Do not use if package is damaged. Destroy after use. The design of this device doesn't allow its reuse or a new sterilization.

These devices should only be used by fully qualified personnel.

Technical procedure:

Connect the catheter to a vacuum system with a depression corresponding to 150 cm of water.

Remove the end of the catheter and pull up the folds of the sleeve to the funnel.

Open the cap of the swivel connector. Insert the distal end of the catheter into the cannula or intubation catheter. Carefully remove the catheter from the sleeve and push it into the intubation catheter or tracheostomy cannula without aspirating. The sleeve is gradually pulled up to allow an aseptic progression of the catheter. Proceed to the secretions aspiration by raising the catheter while maintaining the end of the sleeve and the swivel connector.

Discard the catheter and rinse the suction tube after use.

Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture.

ADVERSE EVENTS

None adverse event known.

SHELF LIFE / PREVENTION / ELIMINATION

Recommended storage conditions: away from moisture and direct heat.

Do not use after expiration date.

Any device used or started must be disposed of an appropriate incinerator in accordance with local or national regulations concerning the disposal of hospital waste.

SYMBOLS USED ON LABELLING

	Consult instructions for use
	Date of manufacture
	Do not reuse
	Use by date
	Batch code
	Manufactured by
	Sterilized using ethylene oxide
	Single Sterile Barrier System
	Do not Resterilize
	Pyrogen Free
	Product reference
	Caution, consult instructions for use
	Do not use if package is damaged
	Temperature Limit
	Keep Away from Sunlight
	Keep Dry
	Medical Device
	Entity distributing the Medical Devices into EU

CE 0123

CE mark, products comply with the essential requirements of the 93/42/CEE European Directive relative to medical devices.

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Rev. 04-21/08

Doc. No. Peters Surgical	Date : 13-08-19	Design by	Checked by	Approved by
IFU Design Front Side for Suction Catheter 65cm with protective sleeve	Size : 210x297mm P. Area :			
Color Break Black	Specifications : 1. 2.	Designer Issue No. 05	QA Person Scale : 1:1	QA Incharge Sheet 1 of 1
Rev.	Amendment		Remarks	Date of Amdt.
1.	Added symbol of Manufactured By			05-09-19
2.	Added some required details, i.e. storage condition etc.			11-02-20
3.	Updated new CE 0123			21-05-20
4.	Added Peter's new logo & address and MD symbol			27-08-21

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Rev. 04-21/08

Doc. No. Peters Surgical	Date : 13-08-19	Design by	Checked by	Approved by
IFU Design Back Side for Suction Catheter 65cm with protective sleeve	Size : 210x297mm P. Area :			
Color Break Black	Specifications : 1. 2.	Designer Issue No. 05	QA Person Scale : 1:1	QA Incharge Sheet 1 of 1
Rev.	Amendment		Remarks	Date of Amdt.
1.	Added symbol of Manufactured By			05-09-19
2.	Added some required details, i.e. storage condition etc.			11-02-20
3.	Updated new CE 0123			21-05-20
4.	Added Peter's new logo & address and MD symbol			27-08-21

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POLY MEDICURE LTD.