## PRODUCT INFORMATION DATA SHEET

Please read the following information thoroughly before using this device.

Product Ref: Re Usable Clip Applier.

Codes: Clip applier- LOK6APPLIERNML/L/XL

Clip cartridge: LOK6LMM002/LOK6LML003/LOK6LMLXL004

• Description:

The Lok 6 ligation system consists of reusable clip applier and a polymer clip cartridge. The reusable clip applier consists of a rotatable shaft, a jaw, and an ergonomic handle. There are three sizes of the polymer clip cartridge for selection including M/L, XL and XL.

The Lok 6 reusable clip applier M/L/XL Nor-sterile Device (they are supplied as a non-sterile device) hospital use their own sterilization protocols to clean them.

Clip cartridges are supplied as a sterile product.

• Intended Use:

The Lok 6 endoscopic reusable clip applier is used for the ligation of vessels, ducts, and other tubular structures laparoscopically.

• Contraindications:

1. Lok 6 endoscopic reusable clip applier is not intended for contraceptive tubal ligation, but may be used to achieve hemostasis following resection of the fallopian tube.

2. Do NOT use the clips for the ligation of the renal artery during laparoscopic living-donor nephrectomies.

3. Do NOT use the clips as fallopian contraceptive tubal occlusion device.

• Instructions for Use:

1. Always, always check the alignment of the applier jaws. Jaw tips should be directly aligned when closed to prevent patient injury. Examine the applier for bent, broken, cracked, or worn parts. Do not use the applier when any kind of damage is observed.

2. Insert the jaws of the applier into the slot of the cartridge. The jaws should be perpendicular to the base of the cartridge. Gently press the applier over the clip until there is an audible double click. Make sure that the clip bases seat in the notches of the applier jaws.

3. Close the applier handles and insert the applier through the cannula. Maintain compression on the handles until the jaws pass through the cannula.

NOTE: DO NOT apply lateral pressure on the applier as damage to the working tip may occur. Always confirm that the clip remains in the applier after insertion of the applier and clip through a cannula. Do not squeeze the trigger without a clip properly loaded into the jaws.

4. The jaw (2) can be rotated for maximized visualization, but the rotation knob should only be activated with jaws in open position.

5. Press the trigger (3) and move the jaws to ligature. To load the clip, squeeze the trigger (3) to the full.

NOTE: The vessels or structures to be ligated should be within the confines of the jaw and no torque or twist of the jaw is allowed. No clip can be placed across another clip.

6. Release the trigger (3) to disengage the clip. It should be confirmed that the clip is closed properly after the ligation.

7. Remove the Lok 6 endoscopic reusable applier from the trocar.

• Transportation:

During the transportation of the instrument to the processing site, store contaminated instrument securely in a closed container to avoid damage to the instrument and/or contamination of the environment.

• Cleaning, disinfection and drying instructions:

Clip applier should be cleaned independently according to the instruction listed below. The instruments are approved to be cleaned according to the following options:

Manual processing instructions for use with Enzymatic Detergents		
Pre-cleaning:		
1. Soak the instrument in tap water for a minimum of 10 minutes.	Note: the preparation of the concentration, temperature and application time of the cleaning agent must be according to the instructions for use provided by the detergent manufacturer. Detergents must be approved for use with metal reusable general surgical instruments.	
2. Disassemble the instrument (if applicable).		
3. Brush under tap water with a soft bristled brush for a minimum of 40 seconds or until all residues are removed.		
4. Flush the instrument with water and proximal handle joint with a syringe until the rinse water is clear.		
5. Rinse and flush with water.		
Drying:		
6. Dry the outside of the instrument using the drying cycle of the cleaning equipment. If necessary, instruments can be dried by hand with a sterile lint-free cloth. Compressed, filtered air can be used for instruments with lumens.		
Cleaning:		
7. Place the instrument in an ultrasonic bath at 104 degree Fahrenheit(40 degrees Celsius) with an enzymatic detergent.		
8. Soak for 3 minutes.		
9. Remove the instrument and rinse and flush with de-ionized water for approximately 40 seconds.		
Disinfection:		
10. Located does not provide disinfection instructions due to the sterility instruction provided.		

Automatic processing instruction for use with enzymatic detergents		
Pre-cleaning:		
1. Soak the instrument in tap water for a minimum time of 10 minutes.	Note: the preparation of the concentration, temperature and application time of the cleaning agent must be according to the instructions for use provided by the detergent manufacturer. Detergents must be approved for use with metal reusable general surgical instruments.	
2. Disassemble the instrument (if applicable).		
3. Brush under tap water with a soft bristled brush for a minimum of 40 seconds or until all residues are removed.		
4. Flush the instrument with water and proximal handle joint with a syringe until the rinse water is clear.		
5. Rinse and flush with water.		
Cleaning:		
6. Place the instrument in an ultrasonic bath at 104 degree Fahrenheit(40 degrees Celsius) with an enzymatic detergent.		
7. Soak for 10 minutes.		
8. Remove the instrument and rinse and flush with de-ionized water for approximately 40 seconds.		
Disinfection:		
9. Located does not provide disinfection instructions due to the sterility instruction provided.		

Note: It is the responsibility of the health care facility to ensure adequate cleaning of the brushes between each use.

• Functional Check:

Inspect instrument for damage and sterilization cycle end prior to each use as they may be damaged during transit to the customer, during storage at the customer's site, during use in previous procedure or during sterilization process. All moving parts must be inspected for wear and confirmed to be functional. Confirm smooth operation during opening and closing of instrument handle and reinforcing screw. Inspect instruments for rust, pitting, cracking or burns, staining or discoloration, and worn or broken parts. Repair or replace any instrument found not to be acceptable.

• Maintenance, handling and operation surgical instruments =

The assembly and disassembly of the instrument must only be performed by personnel trained to the institution's training requirements. The instrument must be cleaned, lubricated, functionally checked and sterilized prior to each use. Use a non-silicone, water based lubricant prior to sterilization. Do not use mineral oil, petroleum or silicon-based lubrication products. Do not use an instrument with broken, cracked or worn parts. Only use the instrument for its intended use. Misuse or improper handling may result in damage, potentially resulting in user or patient injury.

• Sterilization:

Instruments should be sterilized upon the MHRA Guidance of sterilization and cleaning (Memorandum 01:01: Management and decontamination of surgical instruments as follow:

High temperature steam		
Sterilization temperature in Celsius	Maximum allowable temperature in Celsius	Exposure time period
70	75	100 Minutes
80	85	10 Minutes
90	95	1 Minute

Time /Temperature bands meeting the requirements of an acceptable A0 of 400 are as follow:

• Disinfection:

LocoMed does not provide disinfection instructions due to the sterility instruction provided.

• Nettoyage:

LocoMed Lmtd. Unit 4 The Enterprise Centre, Croydon Business Park, Alton Road - Farnham, Surrey - GU10 5EH United Kingdom Tel: +44(0)1252413051 Fax: +44(0)1252727916 www.locomed.com

Distributed by: PETERS SURGICAL Immeuble AURELIUM - 1 Cours de l'Ile Seguin - 92100 BOULOGNE-BILLANCOURT - FRANCE TEL: +33 (1) 48 11 62 62 - FAX: +33 (0) 1 48 11 22 99 E-mail: peter@peters-surgical.com

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