INSTRUCTIONS FOR USE

RECTAL CATHETER

Materials Used

- Poly Vinyl Chloride (PVC)

Indications

- To clear the secretion from the rectum
- Introduction of enema

Contraindications

- Not to be used in patients with known hypersensitivity to any of the materials used

Directions for Use

- Clean the anus of the patient with mild antiseptic solution
- Put some antiseptic jelly on the tip of the catheter without touching the tip of the catheter
- Advance the catheter into the rectum
- Once the catheter is inserted, collect the secretion into a suitable container

Cautions

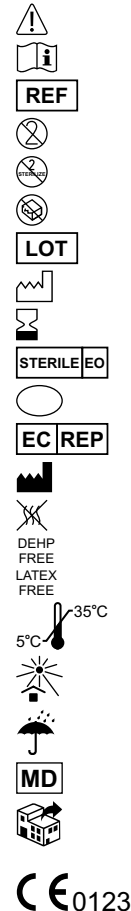
- Do not use if package is opened or damaged
- Aseptic techniques should be followed
- The product should be used only by qualified healthcare professionals
- Not to be used for treatment of Children and Pregnant or nursing women.
- Do not re-sterilize. Discard the set after single use
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines
- Store in between 5°C to 35°C, avoid excessive heat, protect from direct sunlight and moisture
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product

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POLY MEDICURE LIMITED
Plot No. 104-105, Sector 59, HSIIDC Indl. Area,
Ballabgarh, Faridabad-121004 (Haryana), INDIA

EC REP Obelis s.a.,
Boulevard Général Wahis 53,
1030, Brussels, BELGIUM.
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net

Cautions

- Consult Instructions for use
- Catalogue reference
- For single use only
- Do not Resterilize
- Do not use if packaging or product has been damaged or contaminated
- Batch Number
- Date of Manufacturing
- Use by / Expiry date
- Sterilised by Ethylene Oxide
- Single Sterile Barrier System
- European Authorised Representative
- Manufactured by
- Pyrogen Free
- DEHP FREE
- LATEX FREE
- Temperature Limit
- Keep Away from Sunlight
- Keep Dry
- Medical device
- Entity distributing the Medical Devices into EU
- CE marking: The product is in conformance with the basic requirements of European Directive 93/42/EEC concerning Medical Devices

Rev. 03-21/08

Note : IFU will be put inside the duplex box

Doc. No. Peters Surgical	Date : 08-11-19	Design by	Checked by	Approved by
IFU Design for Rectal Catheter Front Side	Size : 180x130mm P. Area :			
Color Break Black	Specifications : 1. 2.	Designer Issue No. 04	QA Person Scale : 1:1	QA Incharge Sheet 1 of 1
Rev.	Amendment		Remarks	Date of Amdt.
1.	Updated the cautions and form of CE symbol			02-12-19
2.	Updated new CE 0123			21-05-20
3.	Added Peter's new logo & address, MD symbol & DEHP Free			27-08-21
4.				

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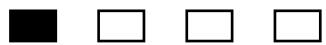
 **POLY MEDICURE LIMITED**
Plot No. 104-105, Sector 59, HSIIDC Indl. Area,
Ballabgarh, Faridabad-121004 (Haryana), INDIA

EC REP Obelis s.a.,
Boulevard Général Wahis 53,
1030, Brussels, BELGIUM.
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net

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Rev. 03-21/08

Note : IFU will be put inside the duplex box

Doc. No. Peters Surgical	Date : 08-11-19	Design by	Checked by	Approved by
IFU Design for Rectal Catheter Back Side	Size : 180x130mm P. Area :			
Color Break 	Specifications : 1.	Designer	QA Person	QA Incharge
Black	2.	Issue No. 04	Scale : 1:1	Sheet 1 of 1
Rev.	Amendment		Remarks	Date of Amdt.
1.	Updated the cautions and form of CE symbol			02-12-19
2.	Updated new CE 0123			21-05-20
3.	Added Peter's new logo & address, MD symbol & DEHP Free			27-08-21
4.				